



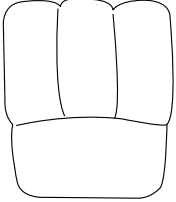

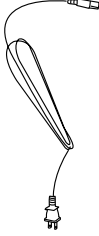

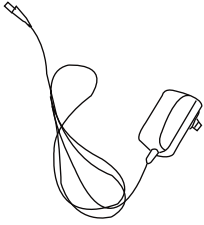
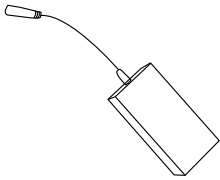


# INSTALLATION GUIDE

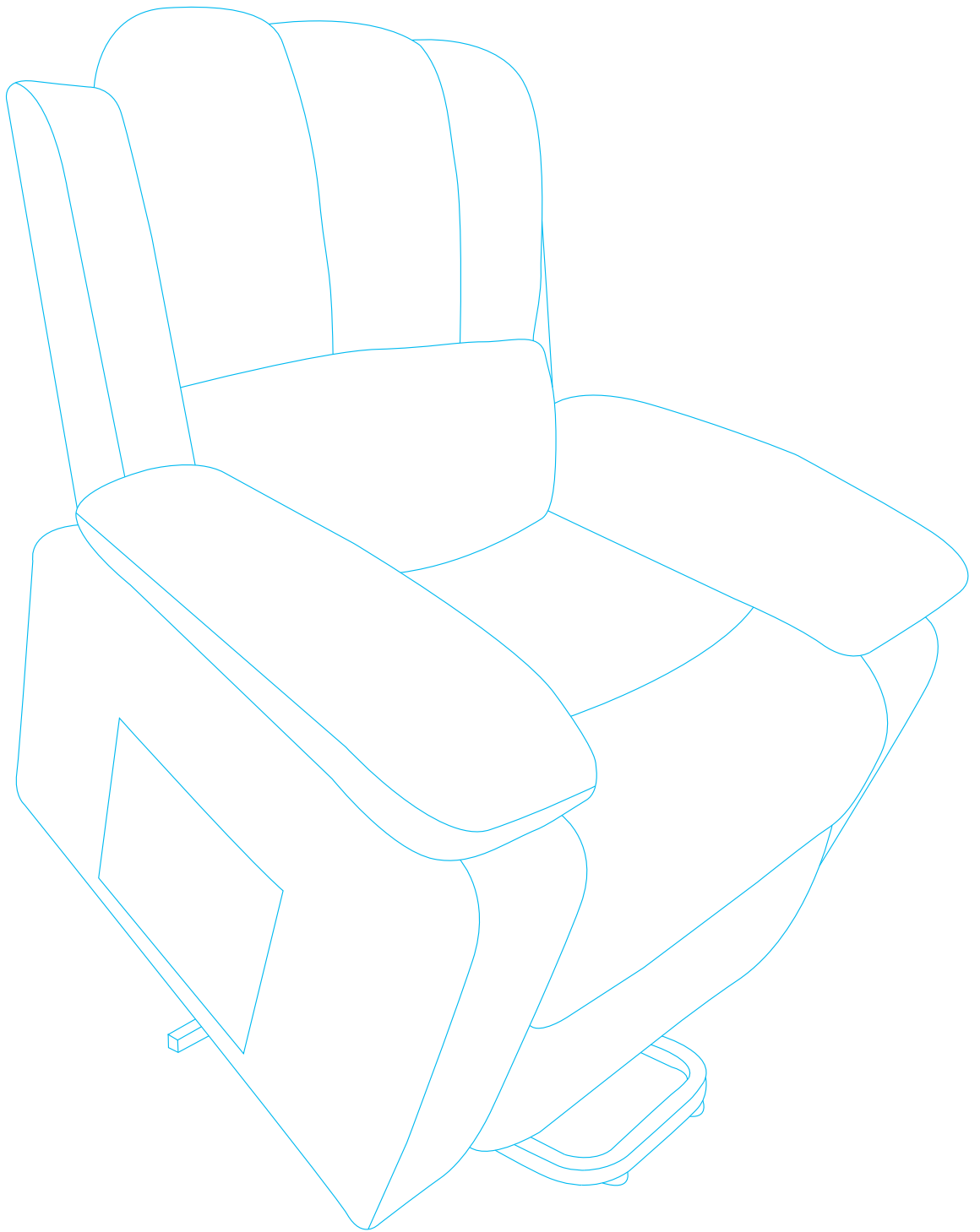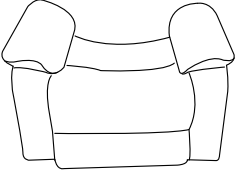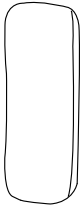


# PARTS LIST

PART	NO.	Qty.
	A	1
	B	1
	C	1
	D	1
	E	1


PART	NO.	Qty.
	F	1
	G	1
	H	1
	J	1

If you find the parts are missing or damaged, please circle the corresponding parts in the picture and contact us by email.



1

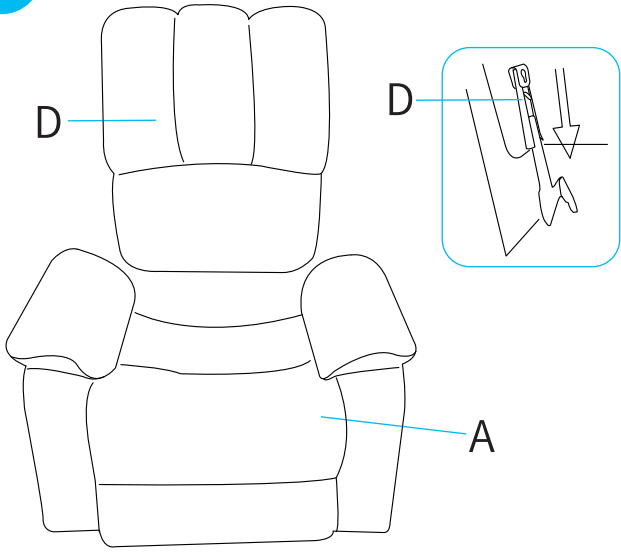


2

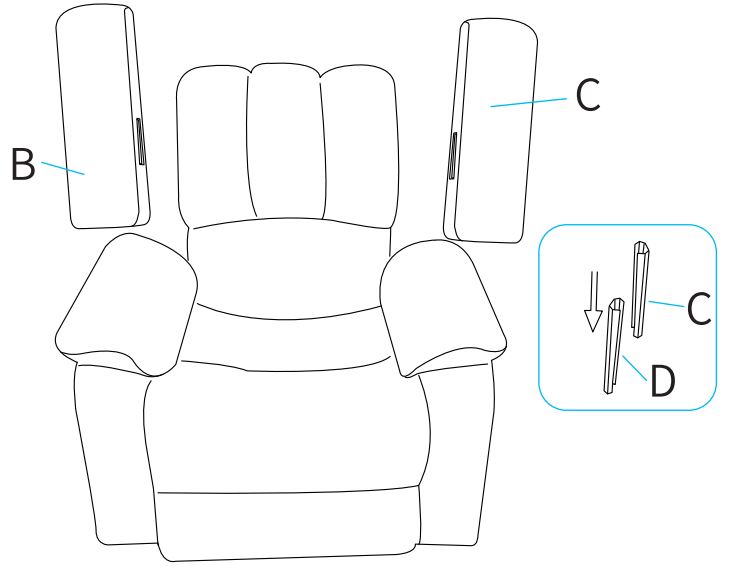


3

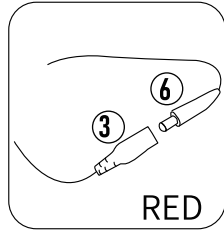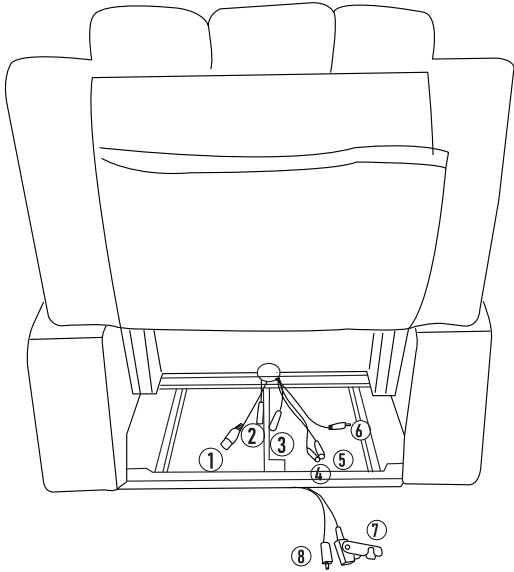


RED

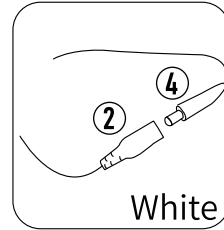


White

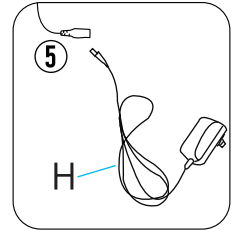

H

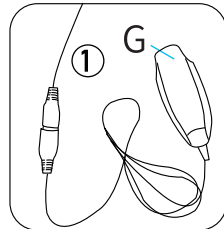


G

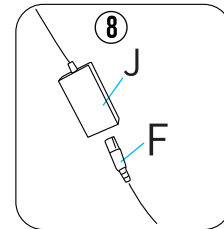

J

F

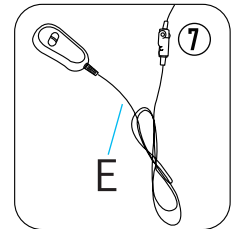


E

4



5

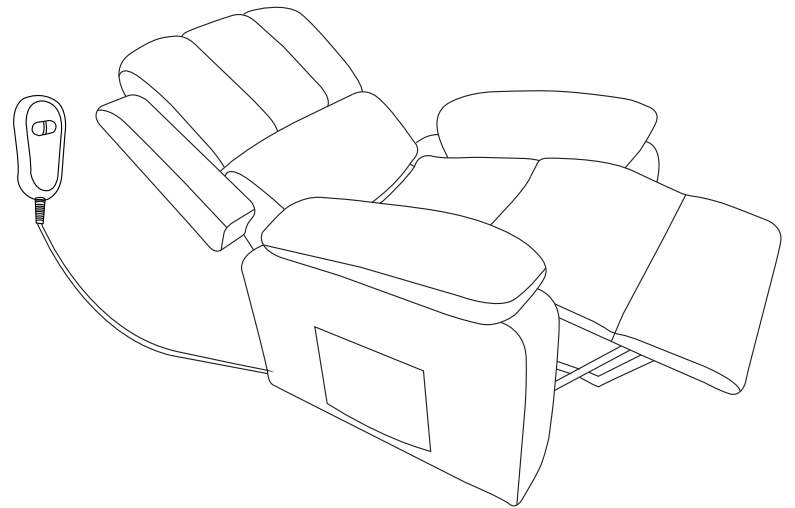

## Usage:

Indoors, avoid high temperature and humid or humid places.

## Tips:

- Do not sit on the remote control or place heavy objects on it.
- Do not drop the remote control on the ground.
- Do not pull the power cord.
- Do not use the remote control with wet fingers.
- Do not press or touch the buttons with sharp objects.
- Never unplug the remote control without turning off the power.

## Troubleshooting:

If the lift chair is not working, check the following chart. Make sure you turn the power off and unplug from the wall outlet. If you have any questions, please contact us. Do not disassemble or repair any part of the lift chair yourself.

### PROBLEM

Noisy during use  
.....

The chair does not function  
when the button on the  
control unit is activated  
.....

### POSSIBLE CAUSE

The noise is caused by  
the internal motor,  
which is normal.  
.....

Check that the plug is  
pushed into the wall  
outlet properly.  
Check whether the power  
supply button is on.  
.....